




- 1-2** Position the NTI™ on patient inferior to the external occipital protuberance and secure by attaching strap to velcro square (*see inset*).
- 3** Open outer tube retaining strap by peeling it away from the velcro to reveal tube securement measures.

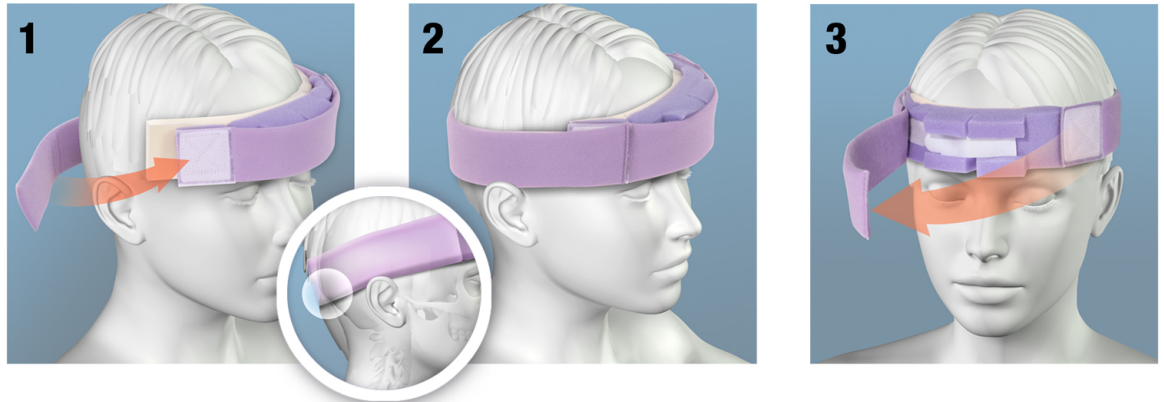

- 4** Open inner tube retaining velcro strap to reveal adhesive tab.
- 5** Peel off plastic liner to expose adhesive (*see inset*).
- 6** Insert tube into slit in foam block and press firmly against adhesive to affix (*see inset*).

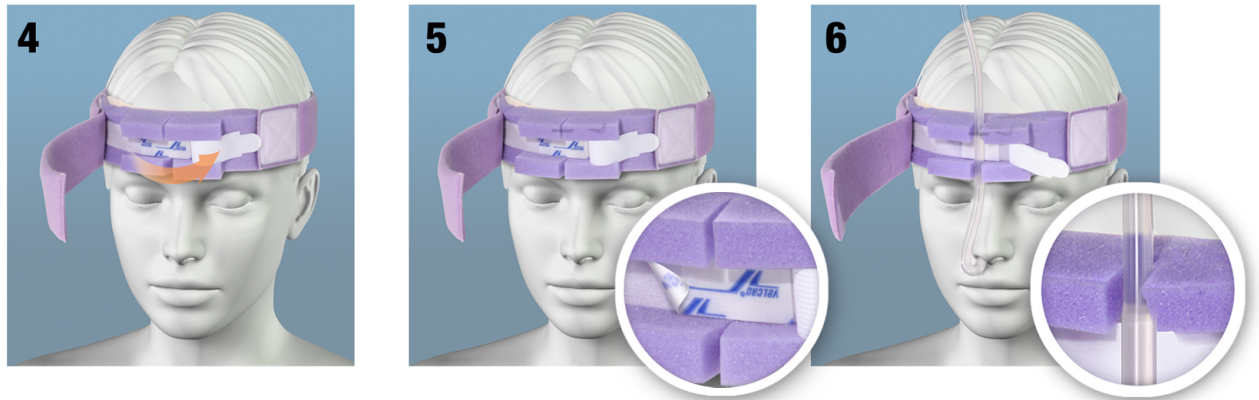

- 7** Secure tube with inner tube retaining velcro strap.
- 8** Tube is now secured in the NTI™.
- 9** Close outer tube retaining strap by securing it to the velcro square. Ensure NTI™ is positioned above and not entrapping the ears and make any final adjustments (*see inset*).

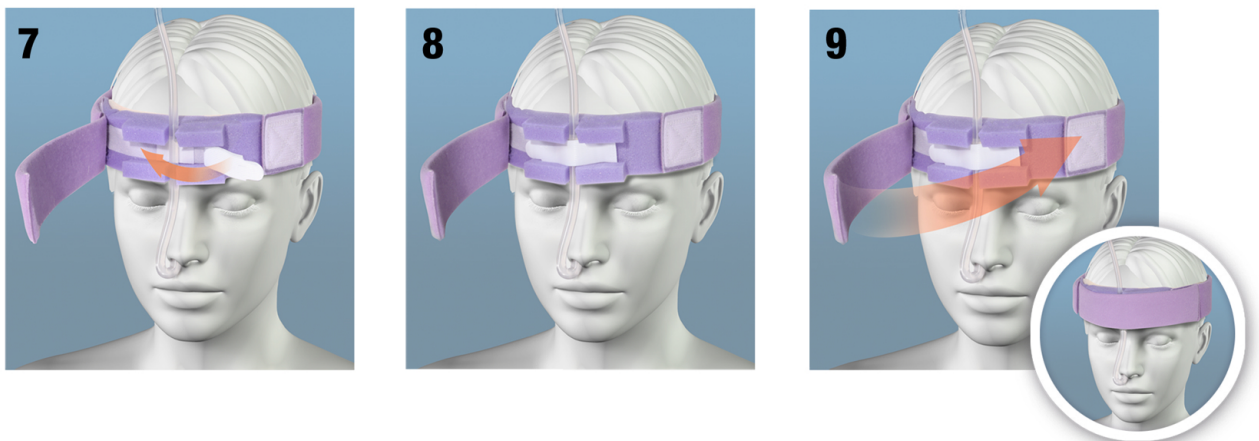

ISO 13485
CERTIFIED



ISO 13485 CERTIFIED
FDA REGISTERED

MADE IN THE
USA



www.xodusmedical.com
info@xodusmedical.com

xodus
medical, inc.

Making Surgery Safer™

Pittsburgh, PA U.S.A • Tel: 724-337-5500
Fax: 724-337-0555